





HIP HOUND MOBILITY BRACE INSTRUCTIONS

FITTING THE HARNESS

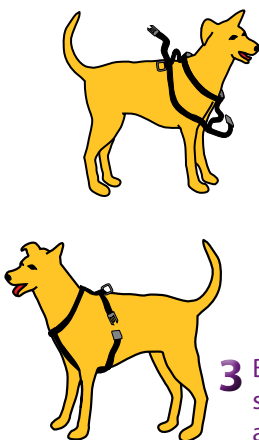

1 With the square ring on top (closest to your dog's tail) and the buckle unfastened, place the dog's head through the loop.


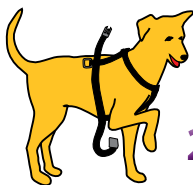


2 Insert the front leg through the shoulder opening.

3 Bring the free strap behind the foreleg and snap the buckle together. Use the slide adjustments to securely fit the harness.


FITTING THE BRACE

Apply the left leg brace first by attaching the leg straps around your dog's thigh, below the hip joint and above the knee joint. Apply the right side in the same manner.

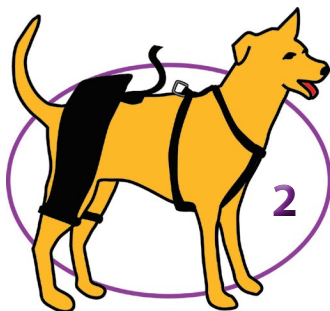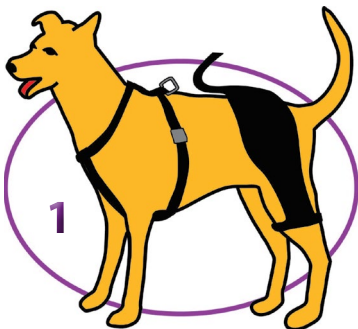

Pull the nylon strap (webbing) through the square ring at the base of the harness and Velcro together. The brace should be secure, but not binding.

Pull the left and right brace firmly up and slightly forward onto the lower back and Velcro the two braces together.

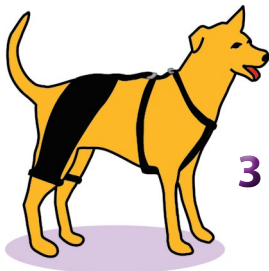



THANK YOU FOR YOUR PURCHASE OF THE HIP HOUND BRACE

While we at Ortho Dog are sorry for your dog's suffering, and need for an orthopedic brace, we are grateful that you've chosen us to help heal and support your special friend.

On the opposite side of this card you will find step by step instructions to help you place the brace on your dog. To further assist, we have created an instructional video which can be viewed on our Web site or on our YouTube channel: youtube.com/user/ORTHODOGBraces1

We hope that the Hip Hound Brace helps your dog regain mobility and improves the quality of life for your dog.

WE LOVE DOGS!

Elizabeth Eve and Dr. Karen Hartman



ORTHO DOG INC.
71 Morton Circle
Swannanoa NC 28778

828.298.0885
www.orthodog.com
bbandzoe@charter.net

Proudly made in North Carolina