




HOCK HOLDER INSTRUCTIONS

The Hock Holder was designed to support and stabilize the hock joint.

The hock joint is on the dog's hind leg, below the knee, and corresponds to the ankle on a human. We designed this brace to fit the natural angle of the hock to prevent it from hyper-extending. The neoprene wrap also supports the joint medially and laterally, allowing your canine friend to move easily and without strain on the joint.

To apply the Hock Holder, simply wrap the neoprene brace around the hock joint on the back leg and fasten securely with the two Velcro straps. The hock joint fits into the hole in the middle of the brace.

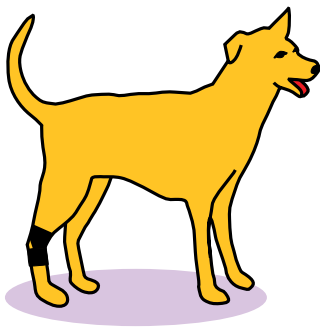

THANK YOU FOR YOUR PURCHASE OF THE HOCK HOLDER

While we at Ortho Dog are sorry for your dog's injury, and need for an orthopedic brace, we are grateful that you've chosen us to help heal and support your special friend.

We hope that these instructions will help you place the brace on your dog and that the Hock Holder helps your dog regain mobility and improves the quality of life for your dog.

WE LOVE DOGS!

Elizabeth Eve and Dr. Karen Hartman

ORTHO DOG INC.

71 Morton Circle Swannanoa NC 28778

828.298.0885 • www.orthodog.com • bbandzoe@charter.net

Proudly made in North Carolina