




# WRIST WRAP INSTRUCTIONS

The Wrist Wrap was the original inspiration for Ortho Dog products.

Your dog's front legs are much like your own arms. Directly above each paw is the wrist joint. The Wrist Wrap, fitted securely, stabilizes your dog's forearm bones and lends support to the muscles and tendons of the wrist and leg.

**To apply the Wrist Wrap,** simply wrap the neoprene brace around the front leg about a 1/2 inch above where the wrist bends, then securely fasten with Velcro. Be sure the brace does not interfere with the flexion of the paw.

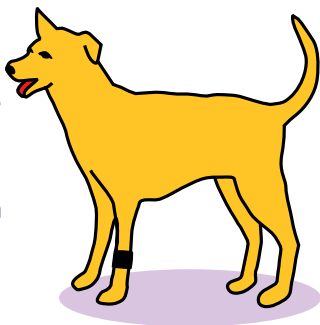

## THANK YOU FOR YOUR PURCHASE OF THE WRIST WRAP

While we at Ortho Dog are sorry for your dog's injury, and need for an orthopedic brace, we are grateful that you've chosen us to help heal and support your special friend.

We hope that these instructions will help you place the brace on your dog and that the Wrist Wrap helps your dog regain mobility and improves the quality of life for your dog.

**WE LOVE DOGS!**

Elizabeth Eve and Dr. Karen Hartman

ORTHO DOG INC.

71 Morton Circle Swannanoa NC 28778

828.298.0885 • [www.orthodog.com](http://www.orthodog.com) • [bbandzoe@charter.net](mailto:bbandzoe@charter.net)

**Proudly made in North Carolina**