




Hip Hound Mobility Brace

INSTRUCTIONS FOR USE



Thank you for purchasing an Ortho Dog brace for your dog. For additional help, watch our step-by-step instruction video available online: orthodog.com/article/how-to-hip-brace

FITTING THE HARNESS

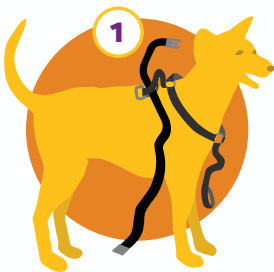

Unbuckle the harness chest strap and slide your dog's head through the neck loop. The square ring should be positioned on top, pointed towards your dog's tail.

Bring the front center strap between your dog's front legs, and slip the long chest strap through the loop at the end.

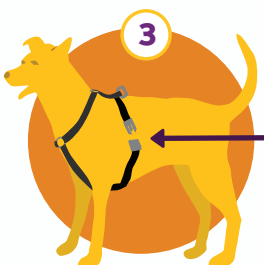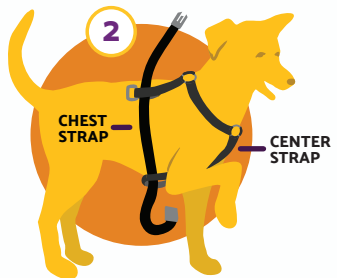


Connect the ends of the chest strap together and use the slide adjustments to securely fit the harness. Two fingers should fit comfortably between your dog and the harness.

FITTING THE BRACE



Apply the Left Leg Brace

First, slide your dog's left leg through the adjustable leg straps on the left leg brace. The brace should sit securely below the hip joint and above the knee joint. Next, slide the top nylon connecting strap through the rectangular ring at the base of the harness, pressing the Velcro firmly together.

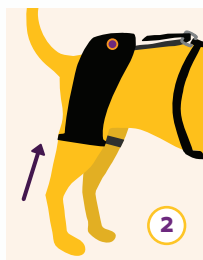

Apply the Right Leg Brace

Slide your dog's right leg through the adjustable leg straps on the right leg brace. Adjust the Velcro straps for a secure fit around the right thigh.



Secure Braces Together on the Lower Back

With the left and right brace pulled firmly up and slightly forward onto the lower back, Velcro the two braces together. The right brace, with the Ortho Dog logo, will be positioned above the other brace.

Helpful Tip!

Once the brace is adjusted correctly to fit your dog, you can open the Velcro connecting strap from the leg portion to the harness, and slide the brace off and on like a pair of pants.

