




# Wrist Wrap & Hock Holder

## INSTRUCTIONS FOR USE

### WRIST WRAP

The original inspiration for Ortho Dog products, the Wrist Wrap stabilizes your dog's forearm bones and lends support to the muscles and tendons of the wrist and front leg.

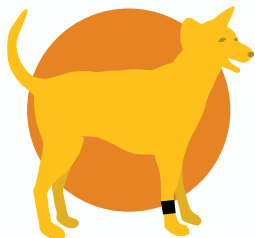

#### How to Apply

Wrap the neoprene brace around the front leg about a ½ inch above where the wrist bends, then securely fasten with the Velcro. Be sure the brace does not interfere with the flexion of the paw.

For additional help, watch our step-by-step instruction video available online: [orthodog.com/article/how-to-wrist-wrap](https://orthodog.com/article/how-to-wrist-wrap)

### HOCK HOLDER

The hock joint is on the dog's hind leg, below the knee, corresponding to the ankle on a human. This brace fits the natural angle of the hock to prevent it from hyper-extending. The neoprene wrap also supports the joint medially and laterally, allowing your canine friend to move easily and without strain on the joint.

#### How to Apply

Wrap the neoprene brace around the hock joint on the back leg and fasten securely with the two Velcro straps. The hock joint fits into the hole in the middle of the brace.



For additional help, watch our step-by-step instruction video available online: [orthodog.com/article/how-to-hock-holder](https://orthodog.com/article/how-to-hock-holder)